Vivaglass® CEM PL

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English

Description

Vivaglass CEM PL is a self-curing, radiopaque, manually-mixed glass ionomer cement with the following properties:

- Good bond to tooth structure
- Low solubility
- Good flow properties
- High compressive strength
- Fluoride release
- Low film thickness
- Universal shade

Dosing = 1:1

1 drop of liquid + 1 scoop of powder

Composition

The powder contains ionomer glass (72 wt%), polyacrylic acid (28 wt%) and pigments (< 0.1 wt%). The liquid contains tartaric acid in an aqueous solution and parabens (< 0.3 wt%).

Indications

- Cementation of inlays, onlays, crowns and bridges fabricated of metal. It is used in those cases, where adequate mechanical retention is present.
- All-ceramic crowns and bridges made of high-strength ceramics (e.g. lithium disilicate glass-ceramics (LS₂), oxide ceramics)
- Metal root canal posts and indirect metal post-and-core complexes
- Orthodontic bands
- Ready-made steel crowns

Contraindication

- Vivaglass CEM PL should not be used if the patient is known to be allergic to any of the ingredients listed.
- Do not apply the material directly onto areas near the pulp or onto the exposed pulp, as irritation may occur.

Vivaglass CEM PL is not suitable for the cementation of

- indirect composites,
- non-reinforced glass-ceramics (e.g. IPS Empress),
- orthodontic brackets,
- ceramic inlays, onlays and veneers.

Side effects

None known to date.

Application procedure

Preparation

- Condition the restoration according to the instructions of the manufacturer (e.g. sandblast contact surfaces of metal restorations). If necessary, degrease with alcohol or acetone.
- Coat dentin areas near the pulp with a calcium hydroxide liner. Clean the prepared teeth with prophy paste (e.g. Proxyt®); rinse with water and dry with an air syringe. Avoid drying out the dentin surface!
- Keep the operating field dry with cotton rolls and a saliva ejector.

Dosing/Mixing

The mixing ratio for a crown is as follows:

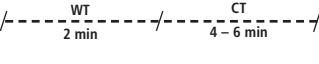
1 drop of liquid and 1 scoop of powder (corresponds to a weight ratio of 3:1).

For the cementation of a bridge, the material quantities are multiplied by the number of abutment teeth.

Dispense powder and liquid next to each other on a mixing pad or a glass plate. Hold the bottle containing the liquid perpendicular when dispensing and squeeze gently to extrude one drop. Shake the powder before use. Overfill the measuring scoop with powder and wipe off excess on the plastic insert in the bottle neck. Close the powder bottle and the liquid bottle carefully after use. On the mixing pad, mix the powder and liquid to a homogenous, creamy consistency with a plastic spatula.

Working and curing times (WT/CT)

At room temperature (23 °C ± 1 °C/ 73 °F ± 1 °F), the following times apply after mixing (mixing time approx. 30 sec):

--

Higher temperatures decrease the working time; low temperatures increase it.

Cementation

Place Vivaglass CEM PL in the restoration and apply the material to the restoration walls with a spatula. Introduce it into the root canal using a lentulo spiral or apply it to the orthodontic band. Excess material must be applied in every case.

- Place the restoration applying light pressure (e.g. with a condenser).
- During setting, the restoration should be kept seated with firm pressure.
- Remove excess cement prior to complete setting of the cement, e.g. using a curette.

If a retraction cord has been used, it should be removed at this time.

Warning

The powder and liquid components and the uncured material may cause slight irritation. Avoid contact with the skin, mucous membrane and eyes. If contact with the eyes occurs, immediately rinse with plenty of water and seek medical advice. After accidental contact with the skin, wash the affected areas with copious amounts of water.

Storage

- Vivaglass CEM PL must not be used after the expiration date.
- Protect powder from moisture. Therefore, carefully close powder bottle after use.
- Storage temperature 12–28 °C (53-82 °F)
- Shelf life: see expiry date

Keep the material out of the reach of children!
For use in dentistry only!

Date information prepared: 09/2012; Rev. 1

Manufacturer

Ivoclar Vivadent AG, FL-9494 Schaan / Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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